Wydanie: 21.09.2021
Edition: 21.09.2021
Ausgabe: 21.09.2021
Vydání: 21.09.2021

Producent/Manufacturer:

 MDH Sp. z o.o.
ul. Maratońska 104
94-007 Łódź
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.qmedbymejra.pl

POZOR!

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EN

Employment according to purpose and indications:

lavicle suport of wide therapeutic use. It effectively helps rehabilitation and treatment in case of collar bone injuries and facilitates correction of postural defects. It can be also used as a preventive aid to establish the habit of correct body posture.

Indications:

Collar bone injuries which do not require surgical and orthopedic intervention. Postural defects which do not require surgical intervention. Pain syndromes of thoracic spine. Thoracic hyperkyphosis.

Contraindications:

Skin abrasions, recent wounds and other skin problems.

Product description:

The support consists of bands made of knitted fabric laminated with foam. Special construction and wide range of adjustment possibilities allow to correct the body posture in patients at different ages.

Use:

Unfasten the brace and put it on - the dorsal pad should be placed between the shoulder blades. Adjust the tapes choosing desired pressure. Fasten the tapes to the shoulder part.

Size: universal

Maintenance:

The product should be washed up by hand using gentle detergents in lukewarm water. Do not chlorinate, do not dry clean, do not whirlpool, wring gently before drying. Do not dry close to the artificial warmth source. Dry in horizontal position. Do not iron.

ATTENTION

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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