Wydanie: 21.09.2021  
Edition: 21.09.2021  
Ausgabe: 21.09.2021  
Vydání: 21.09.2021

Producent/Manufacturer:

 MDH Sp. z o.o.  
ul. Maratońska 104  
94-007 Łódź  
tel. +48 42 674 83 84  
fax. +48 42 636 52 21  
www.qmedbymejra.pl

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EN

**Product description:**

The Genu meyra active knee bandage is a highly effective medical bandage on the latest technical level. The anatomically bi-elastic knitted form offers a very good fit, lateral steel springs provide safety and stability. The bound round elastic silicone ring increases the compression, supports the patella and provides steady support. When moving it has a compression and massage effect on the surrounding tissue. This improves the blood circulation which contributes to the detumescence of the tissue, alleviation of pain, function improvement and relief at the same time.

**Safety instructions :**

The first application of the Genu meyra active knee bandage must be carried out by special personnel trained on these kind of products as otherwise the product liability can not be guaranteed.

**Employment according to purpose and indications:**

The Genu meyra active knee bandage is used in case of slight instabilities and malpositions, arthritis, in case of attrition at the knee joint and chronic irritations.

**Application of the bandage:**

1. Take the bandage at the lateral steel springs from inside and outside and pull it over the knee until the gel pad surrounds the kneecap.
2. Check the correct fit and adjust it if necessary.

**Contraindications:**

Provided that the product is employed properly contraindications are not known. If the bandage is too tight pressure symptoms may evolve. In case of the following clinical pictures the product must only be applied after agreement with the responsible physician: skin changes or inflammations in the area concerned, scarring, hypersensitivity because of heat production, varicose veins, circulatory disturbance in the legs, lymphatic drainage disturbance.

**Application duration:**

The bandage should mainly be worn during physical activity as the function only works in this state. When resting the product should be removed.

**Further information:**

- The active bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Genu meyra active bandage is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

**Storage information:**

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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