Infrapatellární páska

Patella tendon strap

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ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel.

Before the first use, please, read the user's manual carefully.

If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

The strap uses soft knitted fabric and silicon pad to compress the patella tendon. Regulated band allows to choose optimal circumference of the strap and proper compression.

Indications:

Patellofemoral syndrome

Inflammation of patella ligament

Osgood-Schlatter disease

Jumpers Knee

Contraindications:

Necessary lateral stabilization of patella

Injuries of patellar ligament

Gaping wounds, skin abrasion and other skin changes

Use:

Unfasten the strap. Put on so that the silicone pad would be directly below patella, on patellar ligament. Fasten the band the patellar ligament should be compressed but the band cannot restrict circulation.

Maintenance:

Remove the liners before washing. The brace should be washed up by hand using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

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