POZOR!

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**Application:**

Conditions which require immobilization or control of knee range of movement after surgeries or injuries. Treatment and further treatment of knee joint and soft tissue injuries. The product was created to meet the requirements of a commonly accepted process of knee protection and rehabilitation after surgery. Use of cooling gel compress, fits perfectly the „P.R.I.C.E. PROTOCOL”¹.

Contraindications:

States requiring absolute immobilization of the limb. Intra-articular fractures, fractures of tibia, fibula and femur.

WARNING!

! The brace should always be used according to the physician's or physiotherapist's instructions. When used for the first time, the orthosis should first be fitted and matched by qualified medical personnel. The angular range settings should be applied according to the physician's instructions. Please read the manual carefully before using the orthosis. If you experience discomfort while wearing the orthosis, swelling or skin irritation occurs, contact a physician or physiotherapist immediately.

Application:

The orthosis is initially blocked at 0°, the length of the orthosis is 49cm. Optimal alignment of the orthosis by skilled medical personnel is essential to ensure proper knee stabilization.

1. Pull the orthosis out of the package.
2. Loosen each of the five latch clips.
3. Slide the stays to the desired width by grasping the hinges. Pic. 1
4. Place the orthosis on a flat surface under the patient's limb so that the stays are approximately symmetrically on both sides.
5. Each of the stays has 5 independent grommets. By pressing the button located at the central point of the grommet, grab it and extend it to the desired length. Repeat the procedure for each of the slider. Pic. 2
6. Drag the straps by grasping the buckles so that the length of the straps in the back of the orthosis is adjusted as shown. Pic. 3
7. Remove the protective cardboard inserts at the right staples. Pic. 4
8. First button up clamps closest to the knee joint. Then wrap the belt using the appropriate compression force. Make sure the center of the hinges overlaps with the center of the knee. Pic. 5
9. Button up the remaining buckles and wrap the straps using the appropriate compression force. The length of the belts can be adjusted according to the drawing. Pic. 6
10. Make sure the orthosis is properly fitted. Warning! If it turns out that you have to lengthen or shorten the orthosis, unfasten the buckle at the selected area and move the grommets to the required position.
11. In order to set the desired range of motion in the joint, release the Quick Lock (c). The blockades enable the orthosis to be fixed at angles 0°, 15°, 30°, 45°. Pic. 7
12. Press and slide the blockades: flexion (b) and extension (a) to set the desired angle. Use clamps to prevent accidental locking. Pic. 7.2

WARNING!

! Remember that the same values must be set on both clocks.

The package should include:

1. Orthosis
2. Cold Pack Cover (optional)
3. Set of clamps
4. Patellar strap
5. User manual

Maintenance:

Remove the padding elements from the stays before washing. Hand wash lining elements and straps with mild detergent. Do not chlorine, do not wash chemically, do not spin, do not twist, gently squeeze before drying. Dry away from artificial heat sources. Do not iron. Keep away from sources of fire or direct sun exposure.

WARNING!

! Lack of proper personal hygiene, improper use of detergents can cause abrasions or other ailments associated with increased sweating and development of the bacterial flora. Do not use warming ointments. Please note that not-washed off detergent residues can cause skin irritation and damage the material from which the product was made.

Size: Universal

1. P.R.I.C.E. - Acronym for English words: Protect, Recovery, Ice, Compression, Elevation. Procedure to prevent or eliminate inflammation and swelling that may negatively affect the healing process of a damaged or injured joint.

DEFINITIVE DECISION CONCERNING THE APPLICATION AND SELECTION OF THE SUPPLY IS SUBJECT TO THE SPECIALIST

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