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USER'S MANUAL
WAVE WALKER H4
VCBP0032
Edition MG 08.2021

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PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasize that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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1. Handle
2. The aluminum support frame
3. The reinforcing frame
4. The telescopic height adjustment
5. Submission button of a walker
6. The reinforcing frame
7. Legs finished with non-slip rubber protection.

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Walking aid designed for disabled people and people locomotor disorders.

The walker is ideal for rehabilitation process and physiotherapy (verticalization, lower limb relief).

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves.

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HEIGHT ADJUSTMENT

- Before using a walker, should adjusted to the user's height by telescopically adjustable feet length.
- To do this, press the height adjustment pin on each leg, and depending on your needs reduce the height of the legs or extend the leg (Illustration 4).

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- It is important to the length of all legs was the same (handles must be parallel to the ground).
- The handle should be at the height that will provide comfort and a sense of security for the user.

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FIXED FUNCTION

- In order to use the walker fixed function, lift it up, move forward and approach it.

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WALKING FUNCTION

- In order to activate walking function should press one of the function change pin and lift up the reinforcing frame so that pin appeared in lower opening of reinforcing frame.
- Then repeat action for second one pin. (Illustration 5.)
- Please stand near by walker, holding the handle with both hands.
- In order to use the device, interchangeably lift up the walker's lateral parts and move them forward and then put feet.

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CARE AND MAINTENANCE

- Clean the walker frame with a damp cloth.
- Remove contaminants on a regular basis.
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly.
- Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic.
- Do not expose the walker on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

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TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the walker (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws, top and rivets are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVIS

MEYRA ČR s. r. o.

Hrusická 2538/5
141 00 Praha 4
tel.: +420 272 761 102
www.meyra.cz