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USER MANUAL
BILLY
3-WHEEL ROLLATOR
VCBK52211
Edition MG 08.2021

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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE**

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Three wheel rollator, equipped with two fixed rear wheels with brake and locking the wheels and the one front wheel swivel. It lets you easily navigate through any surface, inside and outside the building. Handles anatomical made of polyethylene, aluminium frame.

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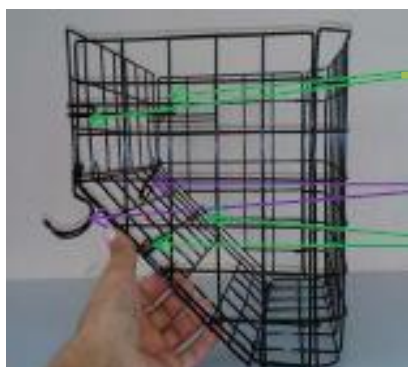
Elements that should be inside the packaging:

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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- Do not use the rollator until all parts are properly connected.
- Installation – Please refer to illustration 1.
- Unfolding rollator should start from unfolded the side frames in the outside of then block the horizontal hinge strut. (Illustration 7)

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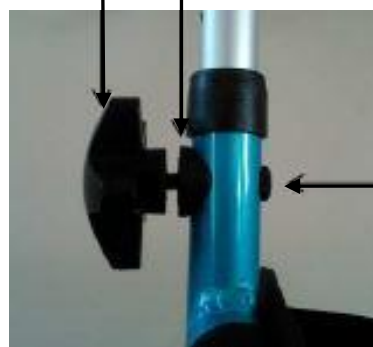
Podložka / Pad

Šroub / Screw

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- In the holes of the side frames should be mounted handle (Illustration 8). From the inside side of frame, should put the screw, while from the outer side, should put pad and then tighten the knob (Illustration 9).

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HEIGHT ADJUSTMENT HANDLES

- Handles should be located at a height that will provide comfort and a sense of security for the user. It is important to ensure that both hands are positioned at the same height.
- To change the height of the handles unscrew the knob for fixing handles, remove washers, the screws and depending on the needs extend or shorten the frame handles.
- After adjusting the appropriate height put the screw on the inner side of the side frame, while the outer side on the screw put pad, and then tighten the knob. (Illustration 8 and 9.)
- Make sure that the handle is properly tightened, so that it prevents movement of the handles.

FOLDING:

- In order to fold rollator for ease of transport, you must first block rotation of the wheels.
- Then it is necessary to remove the tray, basket for small items and emptied a bag for small items.
- Pull up horizontal hinge strut and then submit rollator, directing the handle to each other. (Illustration 12).

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UNFOLD:

- To unfold rollator slide outward side frames, then lock the levels hinge of stretcher.
- Before each use, check the operation of the braking system. Squeeze the brake handles to check if the brake pads are right adhere wheels and stop the rollator.
- Press the handle down to activate the locking rotation of the wheels and check if it works properly.

BREAK FORCE ADJUSTMENT

- To adjust the brake, first, loosen a nut located above an adjusting bolt of the brake just over the wheel.

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BRAKE SYSTEM

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake.
- To use the standard brake, lift up the brake handle.
- To use a parking brake, push down the brake handle to the point where you can hear the distinctive sound of inclusion brake (You should use in that same time both right and left brake).
- To unlock the parking brake, lift up the brake handle back into the basic position.

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OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the rollator.
- Do not expose the rollator on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents.

Do not clean the rollator with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes.	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	Wheel axles.	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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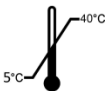
Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVIS

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