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PRODUCT DESCRIPTION:

The shower stool is a light construction made of warm in contact polyurethane and aluminum. Stool legs of regulated height assure stable support and give possibility of asymmetrical position of the stool. There are additional flow openings placed on the stool seat which allows a patient to hold the stool for better balance.

USE

The use of the hygiene procedures using bath or shower with significant limitations of movement of the patient, generally poor condition of the patient, advanced age of the patient.

For use by patients who cannot themselves perform actions and hygiene procedures. Stool protects the patient stable position, protects against the risk of falling.

Protection against wound infection or dipping during hygiene procedures.

Stool can be used in hospital and homecare.

CONTRAINDICATIONS

Contraindications to a sitting position, lack of self-stabilization of the torso, patients with coordination and balance disorders, and with intellectual disabilities should not be left unattended.

STOOL'S EQUIPMENT:

1. Polyurethane seat,
2. Aluminum frame,
3. Telescopic regulation of height,
4. Rubber straps on the top of stool's legs,
5. Openings on the sides of the seat which play the role of handles
6. Colours: silver, white

USE (REGULATION):

The originally packed product consists of separate parts. In order to fold the stool, it is sufficient to turn tight the legs construction to the seat (openings in the legs construction respond to the screwed sockets placed in the seat) – the seat has special grooves which facilitate assembling. Necessary tools: screwdriver.

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SAFETY OF USING:

1. The shower stool is designed to be used on flat, straight surfaces.
2. The use of shower stool on the rough surface (of the slope more than 5%) may result in falling down and injury.
3. It is strictly forbidden to stand on the stool.
4. Maximum load of the stool - 100kg.
5. Don't use the stool in case of observing any visual or detectable damages (such as frame split) because it may result in accident.

SERVICE AND CONSERVATION:

1. Clean the shower stool frame with a moist cloth.
2. Do not use sharp cleaning tools (risk of damaging the protective cover).
3. Clean plastic parts with mild detergent.
4. Regularly clean the walking stick's mobile elements (pawl blockade mechanism).

To maintain functional parameters of the product is extremely important to the maintenance of all its components clean and the manner of storage. Keeping the walking stick outdoor (low temperatures) combined with a lack of hygiene may result in loss of esthetic and functional parameters of the product.

TECHNICAL PARAMETERS:

Parameter	STANDARD	FOLDABLE	ROUND
Height	35 – 54 cm	52 cm	35 – 54 cm
Width	49 cm	49 cm	32 cm
Weight	1,8 kg	3,1 kg	1,22 kg

Materials: Aluminum, polyurethane, rubber.

Serial number/production date: on the product

Durability of product: 5 years from date of sale