# IDENTIFICATION SHEET

Final inspection:

Signature:

Identification tag / Product type (write up):

# patron®

**User Guide** ..... (EN)

- rehab-stroller

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- Reha-Kinderwagen

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EN

### Warranty conditions:

1. The product is covered by 24 months warranty period from the date of sale.
2. The product has to be used entirely to the purpose, to which has been designed for. The user has to follow the instructions mentioned in the User Guide. Producer is not responsible for any defects caused by incompetent handling in conflict with the User Guide instructions.
3. The product has to be used, treated, maintained and stored properly according to the User Guide instructions.
4. Every defect on the product has to be repaired immediately, to avoid any consequential damage. Producer is not responsible for any consequential damage caused by not repaired previous defect.
5. The warranty claim has to be applied at the point of sale (incl. e-shops and alternative sales). Warranty claim procedure follows presently valid standards of EU.
6. User is requested to show undisputable acquiring document of the product to initiate warranty claim procedure. To speed up the process it is recommended to advance the purchase receipt and filled in warranty sheet, including product serial number.
7. Each product is equipped with the ID-label (identification label). If the data on the ID-label are illegible or missing, the user will be requested to proof the identity of claimed product.
8. Warranty claim procedure can be initiated, if the product under claim has been sent securely packed and secured against the damage. It is recommended to send the product in original packaging. Producer is not responsible for any damage caused by the transportation of unsecured goods. The owner is obliged to advance the product under claim properly cleaned up, hygienically flawless due to common hygienic principles. The owner cannot ask the producer to do product basic maintenance or cleaning during claim proceedings.
9. The warranty does not cover the defects caused by regular usage (e.g. upholstery wear-out, pollution or damage, brake and tire wear-out, etc.), forbidden overloading, accident, violence or incompetent intervention into the product or incompetent handling with it.
10. The chassis engineering solution does not assure the ideal straight direction trails.
11. Warranty period cannot be interchanged with the period of use. The period, for which the product, in respect of its characteristics, given purpose and difference in construction, can be used properly, when the product is used and maintained properly according to the user guide.

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Dear customers,  
Thank you for your favour and purchasing of Rehab-stroller DIXIE.

### SAFETY NOTES

**IMPORTANT! Read these instructions carefully before use and keep them future reference. Your child's safety may be affected if you do not follow these instructions.**

**WARNINGS! It may be dangerous to leave your child unattended.**

**WARNINGS! The seat unit is not suitable for a child younger 6 months.**

**WARNINGS! Ensure that all the locking devices are engaged before use.**

**WARNINGS! Any load attached to the handle affects the stability of the pram/ pushchair.**

**WARNINGS! Always use a harness.**

**WARNINGS! Do not use if any of the parts is broken, curved or missing.**

- accompanying person must be familiar with instructions listed in this manual
- the stroller is designed for child up to 30 kgs
- the brakes must be applied every time when the stroller is not moving
- do not over load the stroller, overloading of a seat, a bag or a basket might cause disruption of the stability of the stroller
- always use the front grab bar and safety 5-point harness
- always pay adequate attention to the ride with a stroller to avoid an accident, which could happen in case of carelessness
- when manipulating with the stroller pay attention to not to harm yourself or a child
- be aware of the road traffic regulations when riding the stroller
- accompanying person has to be aware of fire danger; in buildings obtain the information about allocation of the emergency exit and about the evacuation plan of the building
- use ramp or elevator if possible when overcoming obstacles
- the stroller must be transported by at least two persons
- The stroller is not designed to be driven down the stairs.
- never transport the stroller when you and your assistant are not sure if you can transport the stroller or portable baby carrier with the child safely
- getting over the curbstone firstly practice with an empty stroller
- do not use the stroller for other purposes than it is designated for
- do not use the stroller for more children than it was constructed for
- do not use any other accessories, which are not approved or supplied by the producer
- when you are assembling the product often, the structure of the stroller might get curved when the guide for assembling of the stroller is breached
- in case of apparent defect, which obstructs the product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product without delay and contact the dealer.
- company PATRON Bohemia a.s. does not bear the responsibility for the damages caused by using other accessories then supplied by this company
- company PATRON Bohemia a.s. does not bear the responsibility for the damages and accidents caused by inappropriate use of the stroller

### INSTRUCTIONS FOR USE

#### Attaching and removing the seat from the chassis

Attaching: Fix the seat onto "clip- and -go" interface from above, pressure on adapters on each side of the lock. Ensure proper signals by audible click. Pull the seat up, make sure it is correctly locked ( can not be ejected ).

Removing: Press both buttons on the outside of the adapter harness, pulling upwards remove the seat from the chassis.

**WARNING!** Pay attention when carrying the seat!

#### Abductor block

Seat the child into the stroller and find the appropriate place of the wedge. Take the child out of the stroller. Find the relevant fitting hole on the seat base. Place the wedge in the seat (the narrower part should aim to the backrest) and secure the wedge by a nut.

#### Headwings

Set the child into the stroller, to determine right height of headwings. Take the child out of the stroller. Put headwings in determined height onto the backrest, and secure it using straps with Velcro.

#### Assembling the undercarriage

Lay compact undercarriage on the ground, put on all wheels. Remove protective covers of extensible locks on both ends of handle pipes. Stand between the rear wheels of the stroller on the side of the handle. Brake the undercarriage off by pressing the brake pipe to second marginal position, hold your foot slightly on the wheel's axis, grab the handle and pull it upwards, towards yourself, until you hear a click of the fuses. Make sure that the undercarriage is secured by light pushing the handle downwards (the undercarriage cannot be folded now).

#### Unfolding the undercarriage

Brake the undercarriage off by pressing the brake pipe to second marginal position. With your foot on the pipe, grab left and right swivel assembling controller on the handle pipe and rotate it inwards. To assembly the undercarriage to the marginal position, press with your forearm on the handle (until you feel a slight resistance from undercarriage chassis). Ensure with transport safety-catch. Assembled undercarriage can be braked off with your hand, if needed. Do not rest folded stroller by the extensible locks on handle pipes, the folding function could be damaged.

#### Wheels

**Mounting** - press the quick fastening mechanism of the wheel, put the wheel on the axis and release the mechanism. By pulling the wheel make sure, that the wheel is mounted correctly and cannot get loose.

**Releasing** - press the quick fastening mechanism of the wheel and remove it from the axis.

#### Parking brake

To apply the parking brake, press the plastic brake pedal. Please check the brakes regularly and keep good functionality by cleaning and greasing of the mechanism by silicone grease. ( plastic slider inside has to be loosely movable)

#### Height of the handle

The height of the handle can be set by pressing the black plastic articulations of the handle on both sides simultaneously and setting it to the desired position.

#### Stroller operation:

##### Changing the position of the seat:

The seat can be positioned in or against the direction of travel. Apply the brake on the stroller (stroller brake), take the child out of the seat. Grasp the seat firmly, simultaneously press both buttons on the outside of the adapter harness. Pulling upwards remove the seat from the chassis and turn in/against the driving direction. Place onto the "clip- and -go" interface from above, put on adapters on each side of the lock. Ensure proper placement with signals of audible click. Pulling the seat up, make sure it is correctly locked ( can not be ejected ).

##### Positioning of the seat:

You can set four position of the backrest. When installing the backrest unlock mechanism positioning the backrest upwards, adjust it to the desired position and release mechanism with pressure on the backrest downwards you can make sure that the backrest is fixed properly (is not possible to change its position). When you lift the backrest, positioning is working automatically.

### Safety Bar:

To fit the safety bar put both ends of safety bar in seat side tubes at once. To take out the safety bar, press press-pin release and lift the safety bar up.

### Footrest height adjustment:

Hold the footrest. Loosen the footrest height adjusting bolts. Set the position of footrest and tighten properly both adjusting bolts. Turn adjusting bolts small handles into the safest position by pulling it and rotating.

### Footrest set reclining:

Press both press buttons on footrest hinge simultaneously and adjust the angle position of footrest. Free both buttons and set the position of footrest to allow the hinges to be locked.

### Getting over the curbs and low obstacles:

*Down* – hold the handlebar tightly; push the stroller upright with the front wheels to edge of the curb, press a little handlebar down to lift a bit front wheels and then carefully ride off the curb on the rear wheels.

*Up* - push the stroller upright to the curb, press a little handlebar down to lift front wheels to desired height and ride on to the walkway. Hold the handlebar tightly to keep stroller's side stability. Pull the handlebar a little up to lift the rear wheels to desired height and ride on to the walkway. In any case do not bump with the front wheels to the curb!

### Carrying stroller:

Use ramps and lifts if possible. If you have to get over the stairs or other obstacles, the stroller must be carried by 2 persons at least. Never do that if you or your assistant are not sure that you can carry the stroller with child safely. During the carriage child must not have the hands out of stroller.

**WARNINGS!!** NEVER lift the stroller up by holding the grab bar, footrest, seat unit, carrycot or hood of the seat/carrycot!!

### *Down from the stairs:*

Hold the handlebar tightly; push the stroller upright with the front wheels to the top edge of the stairs. Your assistant grabs the chassis above the front wheels. Both of you lift up the stroller, carefully carry that down and set back again on all wheels.

### *Up to the stairs:*

Hold the handlebar tightly; bring upright the stroller backing up to the lower step of the stairs. Your assistant grabs the chassis above the front wheels. Both of you lift up the stroller and carry upstairs so you are going first and your assistant second. Set the stroller back on all wheels in safe distance behind top edge of stairs.

## MAINTENANCE

- If you will comply with below referred instructions, the stroller will serve you unfailingly, and you will not have to use your service department.
- Check locks of the handlebar, safety-catch of the handle, handles on the stroller and joins regularly to keep them in good and functional.
- Treat rotating parts and joins of the stroller by greasing substances aerosol can (e.g. WD 40), minimally every 6 months. Never use oil or Vaseline. Make sure not to use too much of the greasing substance (it would cause dirt sedimentation between the rotating parts). Leftover greasing substance should be cleaned.
- Check regularly the tires inflation (max.170 kPa).
- Folding und unfolding of the chassis should be always smooth. If it is not, clean all joints and then read again proper articles in instructions for use.
- The chassis, plastic parts and wheels can be washed by warm water with addition of common detergent.
- Fabric parts of upholstery wash in hand (maximum temperature 30°C). Do not use laundry dryer. Count with longer drying time.
- Composition: 100% PES (polyester) with water-repellent superficial protection
- Never clean the stroller by abrasive or ammonia based cleaning compounds, bleaches and alcohol based cleaning compounds.
- In rainy weather we recommend to use the rain cover (optional equipment)

## DESCRIPTION

- |   |                                      |
|---|--------------------------------------|
| 1. Handle   | 11. Front wheels                     |
| 2. Handle hight adjustment hinge                                      | 12. Basket                           |
| 3. Transport safety catch   | 13. Lock                             |
| 4. Swivel assembling controller                                       | 14. footrest position adjusting bolt |
| 5. Folding lock   | 15. footrest height adjustment       |
| 6. Rear wheels  | 16. Abductor block                   |
| 7. Wheel quick release button   | 17. Safety Bar                       |
| 8. Brake  | 18. Headwings                        |
| 9. „clip-A-go“ interface for seat unit/carrycot/baby car seat adapter |                                      |
| 10. Brake foot pedal  |                                      |

- In case the stroller was exposed to high humidity atmosphere, wipe it by absorbent material and let it dry in warm place.
- Accessories and fabric components of baby stroller place and store in way that the transparent plastic parts are not folded or turned. They can be damaged if the temperature is 5°C below zero.
- Do not expose the stroller to the long-acting intensive solar radiation.
- Allowed side and horizontal shimmy on the perimeter of tire is + / - 2 mm
- We recommend that after 24 months visit a service department.
- Check regularly the stroller handles, joints and guide lock of handbar to keep them in good working condition
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.
- The stroller can be repaired only by employees of service departments of company PATRON Bohemia a.s. or by specialists authorized by this company.
- Only spare parts supplied by company PATRON Bohemia a.s. can be used for repairs.

Changes on all products of the company PATRON Bohemia a.s. are reserved.

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