IDENTIFICATION SHEET

Final inspection:

Signature:

Identification tag / Product type (write up):

User Guide (EN)

- rehab-stroller (buggy)

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EN

CZ

Warranty conditions:

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The warranty has to be applied in the business organization, where the product was bought, or in authorized service repair shop.
3. **Properly filled Certificate of Warranty and a copy of acquiring document have to be advanced when applying the warranty. Do not pull down the identification card. It has to be readable and undamaged for the whole warranty period.**
5. The product has to be used entirely to the purpose, which it is determined for, the user has to follow the instructions mentioned in the instructions for use. Producer / distributor is not responsible for defects caused by incompetent handling in conflict with the instructions for use.
6. The product has to be used, treated, maintained and stored properly according to directions mentioned in the instructions for use.
7. The warranty proceedings can be initiated, if the product under claim has been sent complete in original package, carefully packed and secured against harm. The owner is obliged to advance the product under claim properly cleaned up, hygienically flawless due to common hygienic principles. The owner cannot ask the seller to do basic maintenance or cleaning of the product during claim proceedings.
8. Every defect has to be claimed immediately
9. The warranty is forfeit if the defects were caused by forbidden overloading, accident, violence or incompetent intervention into the product or incompetent handling.
10. Warranty period cannot be interchanged with using period, i.e. period, for which, when used and treated properly including maintenance, the product, in respect of its characteristics, given purpose and difference in construction, can be used properly.
11. The engineering solution does not guarantee the ideal direction trails.

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PIPER (ver. Comfort / sizes: UNI)

Date of sale (day, month, year):

Name (block letters), phone number, stamp and signature of the seller:

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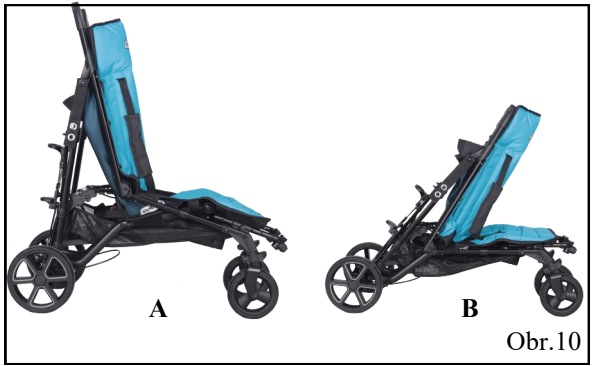
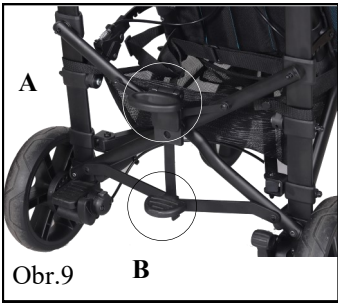
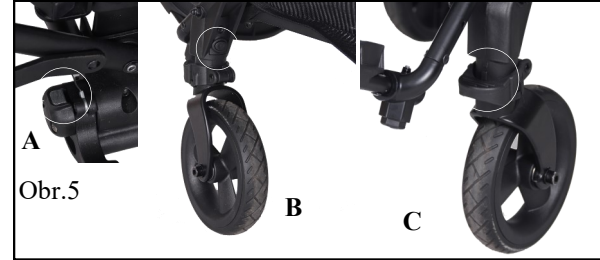
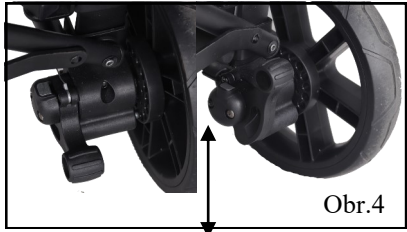
Dear customers,
Rehab-Stroller PIPER is designed for short stay of an occupant. With the design and especially unique construction are ranked among the best from their category. Prior the first usage of the buggy, please, read carefully this user guide. In case you are not fully sure about the proper usage of the product after reading of the user guide feel free to contact your local dealer. Retain the user guide for future reference.

SAFETY AND WARNING NOTES:

① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the stroller!**

.....maximum load.

.....weight of the product

- **WARNING! Keep fingers out of any moving parts when operating with the stroller.**
- **WARNING! Never leave your child unattended! Even if the belts and brakes are in use.**
- **WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.**
- **WARNING! Check before use that seat unit is properly locked in the chassis interface.**
- **WARNING! The seat unit is not suitable for a child younger 6 months.**
- **WARNING! This product must not be used while jogging or in-line skating.**
- **ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.**

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc. are tucked in, so they do not fall in the wheels.
- The stroller MUST NOT be used as a seat for a child while traveling in motor vehicle or bus.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Do not allow the child to climb unassisted into and out from the stroller.
- Always apply the parking brakes, when getting in and out the stroller.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.
- Do not stand on the footrest.
- The brakes must be engaged at all times when the stroller is not moving.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the stroller in two persons.
- The stroller is not designed to be driven down the stairs.
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes then it is designed for.
- The stroller is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.
- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.
- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

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MAINTENANCE

The stroller has been designed to the high standard and with correct usage and maintenance will provide you many years of trouble free service. Regularly inspect handle folding release, safety bar fittings, locking devices, catches, footrest, joints fixings and handle release safety clip to ensure they are secure, clean and in full working order. Brakes, wheels and upholstery of the stroller are expendable parts and should be replaced if necessary by a service centre.

It is recommended that the stroller is checked by an approved service centre every 12 months.

If the stroller is damaged or in need of any repair, only authorized service centres are allowed to provide this services.

Only spare parts supplied by company PATRON has to be used for repairs. Rotating and moving parts of the stroller treat regularly by spray lubricants (e. g. WD 40), minimally every six months. Never use oil or grease.

Ensure that the lubricant penetrates between the components but do not overspray as this may encourage the accumulation of dirt. Wipe off any spray excess.

Folding and assembling the stroller should be always easy. If not, please read again the appropriate parts of user guide.

Check regularly the tire pressure.

Wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.

The upholstery of the stroller can be washed by a cloth wetted in warm water or hand washed (max. 40°C)

Never clean with abrasive, ammonia based, bleaches and alcohol based cleaning compounds.

After exposure to damp conditions, dry the product off with a soft cloth and leave fully open in warm and dry environment.

Store your stroller in a dry, safe place with recommended temperature 5-20°C.

Check and tighten the screw connections regularly.

– We recommend that after 24 months visit a service department.

If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

NOTES

Changes on all the products of the company PATRON Bohemia a.s. are reserved.

SCHEMA DESCRIPTION

1. handle	12. rear wheels
2. folding hinge handle	13. buttons to remove rear wheels
3. safety belt	14. brake lever
4. calf rest	15. pedal to unfold
5. hinge of calf support	16. grip to fold
6. footrest	17. backrest recline locking catch
7. locking pin of the footrest	18. transport safety-catch
8. buttons to remove the footrest	19. backrest
9. front wheels	
10. swivel lock	
11. front swivel castor wheels	

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