ribcap

1. **Instructions for use** English
2. **Gebruiksaanstructies** Dutch
3. **Brugervejledning** Danish
4. **Gebrauchsanweisung** German
5. **Käyttöohjeet** Finnish
6. **Mode d'emploi** French
7. **Istruzioni per l'uso** Italian
8. **Instruksjoner for bruk** ... Norwegian
9. **Instrucciones de uso** Spanish
10. **Bruksanvisning** Swedish
11. **Návod k použití** Czech

1. Instructions for use

1. Safety guidelines

Please ensure the head protector is in a faultless condition before use.
The head protector series bears the CE quality mark and complies with the requirements of:

- 93/42/EEC for Class I medical use - (EU) 2017/745
- PPE1, Directive 89/686/EEC: for minimal-risk lifestyle activities - (EU)2016/425

2. Safety and wearing guidelines.

Always keep out of the reach of unauthorized people!

Adjust the head protector correctly before use!

RIBCAP® head protectors have an adjustable chinstrap and side-releasing safety buckle. Except: Chessy, Bieber, Palmer and some baseball cap series. Non-exhaustive list.

Adjust the straps to ensure a good fit of the head protector, but not too tight.

Please ensure both straps are firmly locked! Make sure the wearer understands how the buckle works before using.

The RIBCAP® head protector may only be used for the indications listed and must not be used for any other purposes (for example as a helmet in general, or for a helmet used for bike riding, horseback riding and other sports)!

Ensure the head protector is always handy, so that it can be put on quickly and properly!

The label inside the RIBCAP® must be at the back when placed on the head.

When the product is not being used, store it in normal indoor conditions. When disposing of the packaging and the product, comply strictly with the legal norms of your community.

3. Operating life

To obtain optimum protection, the product must be treated with due care and diligence. Ensure a precise fit. The RIBCAP® must be replaced after 3 years with a new RIBCAP®, as optimum protection can then no longer be guaranteed due to the deterioration of the material properties. The product must be replaced if required by one or more of the following criteria:

- The fit changes, for example through growth, changes in head shape, etc.
- Exceptional strain to the head protector
- Frequency of wear, frequency of falls
- Degree of care
- Hygienic conditions of the head protector, influenced by (for example) medication and clinical picture.

RIBCAP® guarantees all its products as long as they have not been manipulated or altered from their original state. The guarantee does not apply for products that have been altered from their original state as a result of misuse, deficiencies or breakage of any kind. If you observe any deficiency or anomaly, immediately inform the establishment from which it was obtained in order for it to be changed.

4. Cleaning and care of the head protector

RIBCAP® head protectors may be cleaned using mild soap and a damp cloth or sponge. Strong solvents may contain substances that can damage the head protector and thereby impair its protective effect.

The head protector may be hand washed at a temperature of 30°C. Do not expose the head protector to high temperatures! Washing instructions are on the labels inside the RIBCAP®. To dry the RIBCAP®, leave it to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

5. Re-use

Re-use of the head protector is excluded, it is meant for single patient use.

The head protector should only be used if in perfect condition!

Immediately contact your specialist dealer if there are any defects!

6. Indications

The head protector serves as protection for uncontrolled movements and active lifestyles, senior

care, patients, people with cerebral palsy, epileptic patients, neuromuscular disorders that cause a loss of balance, babies with craniums that are not yet fully formed, after an operation as protection against knocks, spasticity and in all cases where there is a risk of head injury and others where there is a risk of falling and in auto-aggressive patients. Note: RIBCAP® is not certified as a protector or riding helmet (such as skiing, skating, cycling, ... etc.).

7. Purpose of use

RIBCAP® head protector models serve to protect from head injuries caused by uncontrolled falls or sequences of movements, they cover fall hazards with minor or medium potential for injury through the protective effect of the shock-absorbing foam padding.

The top of the head is almost completely enclosed by pads. The head protector comes in different sizes in a range from 47cm to 65cm. (18.5" to 25.6").

8. Accessories

Various options are available on RIBCAP® products at an additional cost. Please contact your local specialist or the manufacturer of RIBCAP® for more information.

9 Fitting instructions

Select the right size by measuring around the head at forehead level. Fit the headgear onto the wearer's head, making sure that the front is correctly positioned, and adjust the chinstrap to the required tightness to ensure that the headgear is stable and secure.

10. Warranty

There is a 2-year warranty on this product. The warranty period begins on the date of purchase (shown on the purchase receipt). During the warranty period, you can submit any complaints about a defective RIBCAP® to your dealer or to the service address specified. In this period, we will repair all defects due to faulty materials or operation free of charge. Improper handling, falls, impacts and the like are excluded from the warranty. A new warranty period does not start if the RIBCAP® is repaired or exchanged; the 2-year warranty period as of the purchase date continues unchanged.

Any warranty expires in the event of non-compliance such as use other than as intended, repairs and alterations made to the head protector or its components by unauthorised people.

Thank you for your interest in our products.

If you have any questions, please contact us or our dealer and sales representative.

Your RIBCAP® Team

RIBCAP® (Glomarket bv, Zwaarveld 34, 9220 Hamme, Belgium) phone +32477 66 96 46, e-mail: contact@Ribcap.com

RIBCAP® is an International Trademark Registration of Glomarket bv (Belgium)

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