1. Úvod

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

Product description

Anti-bedsore cushion consist of numerous, elastic, air interconnected cells provide proper pressure distribution which prevents skin abrasions and bedsores. Inflating and pressure control is possible due to hand pump and air valve.

Cover made of breathable, perforated fabric and empty spaces between the cells provide optimal comfort of use and air flow - warmth is taken away from the body which prevents excessive perspiration. The cushion is easy to maintain, highly hygienic.

Additionally, the cells in two-zones version are divided into two zones. Each of the zone has separated valve. Thanks to such construction its possible (depending on cushion position- transverse or longitudinal) to correct patients position or adjust tilt angle.

Anti-slip pad maintains the cushion on place.

Application

The cushion prevents skin abrasions caused by sitting for long periods, relieves lumbar spine, provides proper pelvic position which prevents pain related to lumbar spine overloading.