PL

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ENG

INTENDED USE:

Follow is a functional knee joint orthosis that efficiently stabilizes the knee joint and protects its ligaments after injuries, surgical procedures, as well as during the period of regaining sports and professional activity.

INDICATIONS:

- decreased post-traumatic capacity of the muscle and ligament structure that stabilizes the knee joint,
- condition after reconstructing the ACL, PCL, MCL and LCL ligaments,
- ACL and PCL ligament failure,
- weakened strength of the quadriceps muscle,
- regaining sports activity after knee joint injuries.

⚠ CONTRAINDICATIONS:

Conditions requiring complete immobilization of the knee joint, hardly healing wounds or skin lesions present within the area covered by the orthotic activity.

DESCRIPTION OF THE PRODUCT:

- anatomically profiled, extremely light aluminium frame,
- flat, four-axis lock with convenient adjustment of the angular range of bending movement (10°, 20°, 30°, 45°, 60°, 90°) straighten (10°, 20°, 30°) and immobilization 0°,
- soft, comfortable, non-slip upholstery,
- stabilising strips ensuring proper placement, as well as efficient operation of the orthosis,

- additional anti-migration strap,
- replaceable condyle pads (under the hinge) for the best fit of the orthosis to the contour of the joint and for comfortable use - prevention in skin irritation and abrasions,
- handy bag for storing added to every each orthose.

USAGE:

When the orthosis is placed for the first time, it should be fitted and put on by qualified medical personnel.

Before placing the orthosis, it is essential to establish proper angular limitations of movement, according to recommendations stated by the physician or the physiotherapist. Make sure that settings on both hinge are symmetrical.

For setting the appropriate angular limit, use the appropriate locks. All limiters are marked with a letter indicating the side on which they should be used, namely L – left or R – right, as well as with the angle of limited movement, next to the indication of limited movement, such as F – flexion and E – extension **3**.

Place the selected limiter of the extension angle range at the front of the hinge and screw it with the help of the attached screw **1**.

Repeat the activity at the back of the hinge, by placing a limiter with indication of the selected range of the flexion angle **2**.

Unfasten all the straps and the anti-migration belt. If the anti-migration strap is not needed, it can be easily removed it from the orthosis.

Place the orthosis in a way, which ensures that all frame elements are resting on the front part of the thigh and on the calf (from the back), and that side rails are placed symmetrically on both sides of the knee joint. The hinge axis should be located at the level of the upper patellar margin **4**. Make sure that the hinge adheres properly to both sides of the knee joint. If not, use proper condyle pads to adjust the hinge.

Fasten the strips in a strictly determined sequence: **4**

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Al, PA, PE, PP, St

CLEANSING:

Clean the frame elements with a damp cloth. Wipe with a damp cloth or wash by hand in temperature not exceeding 35°C, do not chlorinate them, do not dry clean, do not wring, squeeze delicately before drying. Dry away from artificial sources of heat. Dry flat. Do not iron.

⚠ WARNING:

Failure to maintain proper personal hygiene, using inappropriate washing agents may cause abrasions or other ailments associated with increased sweating, as well as development of the bacterial flora. It is worth remembering that remnants of laundry detergent may cause skin irritations and may damage the material that the product is made of.

⚠ WARNING:

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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