Wydanie: 21.09.2021  
Edition: 21.09.2021  
Ausgabe: 21.09.2021  
Vydání: 21.09.2021

Producent/Manufacturer:

 MDH Sp. z o.o.  
ul. Maratońska 104  
94-007 Łódź  
tel. +48 42 674 83 84  
fax. +48 42 636 52 21  
www.qmedbymejra.pl

#### POZOR!

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EN

AFO-FIT is a light, hygienic orthosis AFOs which supports and positions a limb with tight, contracted muscles into a more normal position.

#### Indications:

Prevention of foot-drop in case of peroneal nerve palsy and neurosurgical diseases.

#### Product description:

AFO-FIT is a foot-drop orthosis made of polypropylene. Due to its construction is light and leaves the calf and heel free. Self-fastening shin brace provides basic stabilisation reinforced by additional stiff brace embracing the ankle joint. Rectangular plate ensures optimal comfort of use. AFO-FIT is available in left and right leg version.

#### Putting on:

1. Place the rectangular plate below the shin where the Velcro band is placed.
2. Put the orthosis into the shoe. Put on the shoe and fasten the upper band.
3. Then put the additional stiff brace above the ankle and fasten it adjusting the compression to the individual needs.

#### Conservation and maintenance:

Product hygiene is important for patient's rehabilitation. The brace should be washed with gentle soap and water. It can be also washed with gentle detergents. Please, do not use the strong chemical detergents and bleaches.

#### ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric. The product is not fireproof. Do not use close to the open sources of fire.

#### Size:

Size	S	M	L	XL
Shoe size (cm)	35 – 37	37-39	39 – 41	41 - 44

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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