#### CONSERVATION AND MAINTENANCE:

The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat source. Dry in horizontal position. Do not iron.

#### ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric. The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A CONSULTANT.

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## INSTRUCTIONS FOR USE

### AFO-SOFT – Stabilizing ankle joint orthosis

AFO-SOFT is an open type orthosis of wide therapeutic use. It helps with treatment and rehabilitation of ankle joint injuries and dysfunctions. Its construction supports the lateral ankle ligaments and medial orientation of the foot.

#### INDICATIONS:

- Support in chronic ankle joint instability, treatment of injury of lateral ankle ligaments,
- Follow-up treatment of ankle fracture, follow-up treatment after surgery,
- Recommended in case of joint capsule injury, ligament inflammation, ankle joint instability accompanied by rheumatic diseases,
- It protects against ankle joint hurt after former injuries.

#### PRODUCT DESCRIPTION:

AFO-SOFT is made of breathable Air-X fabric and it is equipped with special "comfort areas" which are placed on vulnerable spots to protect against abrasions and to provide better stabilization. Afo-Soft is fastened with Velcro bands. There are also 8 flexible figures stabilizing straps which allow to regulate the compression strength, maintain bending stress and support collateral ligaments. Available in one, universal size.

#### PUTTING ON:

1. Unfasten the orthosis and put it onto the foot.
2. Fasten the Velcro straps and adjust compression to the individual needs.
3. Fasten the stabilizing straps.

Size	S	M	L
Leg circumference above the ankle (cm)	19 - 22	23 - 25	26 - 28