

Návod k použití
Instructions for use

ZÁVĚS ARM SLING

UNIVERSAL SLING

Katalogové číslo / Catalogue Number:	PHAG01
Basic UDI-DI:	59017804PHAG01HL



Výrobce / Manufacturer:

Mdh sp. z o.o.,
Ul. Maratońska 104, 94-007 Łódź
Tel. +48 426 748 384
www.mdh.pl, www.qmedbymejra.pl

Vydání 15.09.2021
Edition 15.09.2021

Qmed[®]
BYMEYRA

CE

NÁVOD K POUŽITÍ

Univerzální závěs

Univerzální závěs Qmed Pharma je určen k podpoře a částečnému znehybnění horní končetiny, když je nutné končetinu udržet v jedné poloze. Závěs je lehký a pohodlný. Ani při celodenním nošení nezpůsobuje nepohodlí. Výrobek je zvláště užitečný po sejmutí sádry jako dočasný obvaz.

INDIKACE:

Závěs podpírá paži a přenáší váhu na záda v případě výronů, zlomenin nebo pro odlehčení končetiny po operaci. Pomáhá udržet paži v neutrální poloze.

APLIKACE:

Umístěte předloktí do závěsu a poté si přetáhněte podpůrný pás přes hlavu.

ÚDRŽBA:

Perte ve studené vodě, nechte volně uschnout. Nepoužívejte bělidlo. Nežehlit. Nečistěte chemicky.



VÝBĚR VELIKOSTI:

Změřte si délku předloktí.

Velikost	S	M	L	XL
Délka předloktí (cm)	20 – 25	25 – 30	30 – 35	35 - 40



POZOR! Neaplikujte přímo na rány nebo poškozenou pokožku. Kůže v ortéze by měla být čistá, suchá, bez mastí, krémů apod.

Opakovaně použitelný výrobek.

KONEČNÉ ROZHODNUTÍ O APLIKACI A VÝBĚRU VYBAVENÍ PATŘÍ LÉKAŘSKÉMU SPECIALISTOVĚ.

INSTRUCTIONS FOR USE

Universal sling

The Qmed Pharma universal sling is designed for supporting and partially immobilising the upper limb if it needs to be kept in one position. Sling is lightweight and comfortable. It does not cause discomfort even during all-day use. The product is especially useful after the removal of a cast as a temporary dressing.

INDICATIONS FOR USE:

The sling supports the arm by transferring load to the back in sprains and fractures. It can also be used for unburdening the limb post surgeries. It helps keep the arm in a neutral position.

APPLICATION:

Place the forearm in the sling and then pass the supporting belt over your head.

MAINTENANCE:

Wash in cold water, leave to air dry. Do not bleach. Do not iron. Do not dry clean.



SIZE SELECTION:

Measure your forearm length

Size	S	M	L	XL
Forearm length (cm)	20 – 25	25 – 30	30 – 35	35 - 40



REMARKS: Do not use directly on wounds or damaged skin. The skin within the orthosis should be clean, dry, without lotions, creams etc.

Reusable product.

THE FINAL DECISION REGARDING THE APPLICATION AND SELECTION OF EQUIPMENT IS MADE BY A SPECIALIST PHYSICIAN.