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### Support Stoma Waistband

#### INDICATIONS:

- For patients with intestine fistula (especially after surgical period) the waistband reduces the risk of hernia emergence.
- In case when hernia already occurred.
- Waistband strengthens the muscles of stomach during physical activities.
- Waistband can be applied in cases of various fistula location.

#### SIZE ADJUSTMENT:

In order to properly adjust the size, measure the circumference of the abdomen, and then choose the size of the table on the packaging.

Rozmiar	XS	S	M	L	XL	XXL	XXXL
Abdominal circuit (cm)	<56	56-71	69-84	81-96	94-109	107-122	120-135
Height (cm)	25						

#### WARNING!

Packaging does not contain ostomy bags.

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1. Unclip the fastener.
2. Arrange the waistband embracing the abdomen and lumbosacral area, so that the hole for ostomy bag composed in the correct position.
3. Fasten the velcro strap with proper compression force
4. After lashing an ostomy bag protect it with mesh shielding .

#### STORAGE:

In the original container, dry and dark place.

#### MAINTENANCE:

Hand wash using a mild detergent assigned for wool washing. Don not allow heavy soiling, do not iron, do not bleach!

#### UWAGA!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Before use, contact your doctor.