Wydanie: 21.09.2021

Edition: 21.09.2021




Auszgabe: 21.09.2021

Vydání: 21.09.2021

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Producent/Manufacturer:

 MDH Sp. z o.o.  
ul. Maratońska 104  
94-007 Łódź  
tel. +48 42 674 83 84  
fax. +48 42 636 52 21  
www.qmedbymejra.pl

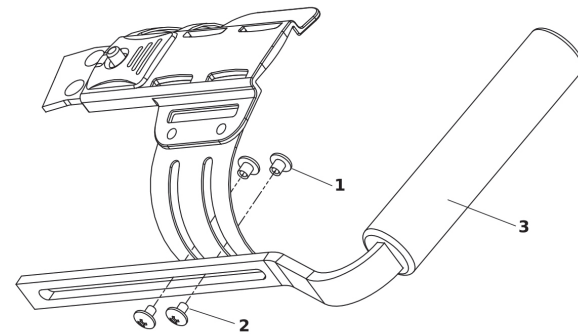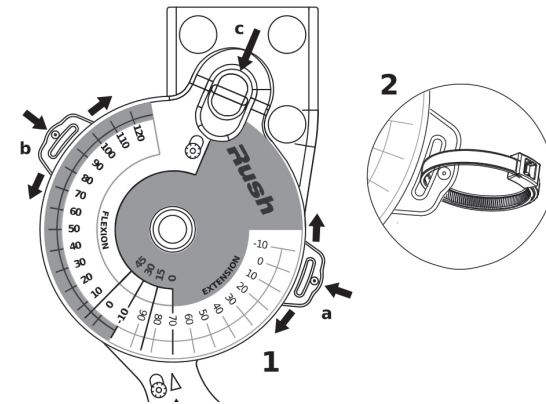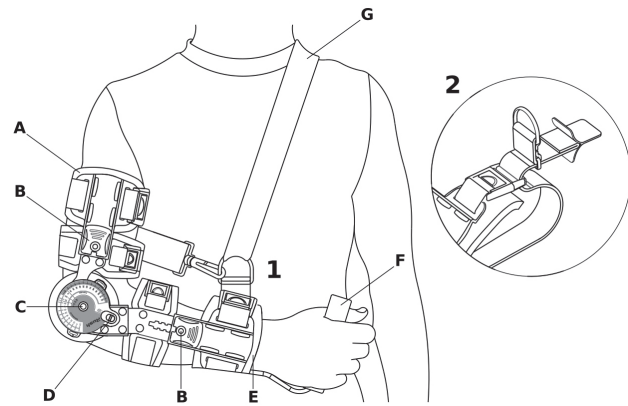**CZ**

#### Indikace:

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**EN****Indication:**

Treatment, follow-up treatment and rehabilitation of joint injuries which require limitation of pronation/supination motions in forearm and motion control in elbow joint.

**Contradictions:**

Conditions which require complete immobilization of elbow joint, difficult to heal wounds, sores and other skin changes.

**ATTENTION:**

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according

to instructions given by doctor or physiotherapist. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please consult a doctor or physiotherapist.

**Product description:**

- A pads prevent sliding and rotation of the brace and provide safe contact with the skin
- B button allows to adjust length
- C hinge enable angular adjustment of motion or elbow joint immobilization
- D Quick Lock with 0°, 15°, 30°, 45° adjustable angles
- E flexible cuff provides the possibility to adjust the brace to the limb circumference
- F hand holder
- G support belt

**Support belt application:**

- Clip the ring that secures the support tape to the wrist tape.
- Attach the tape as shown in the picture.
- Fasten the end of the strap to the fastening ring using the buckle at the end.
- The length of the tape should be adjusted by unfastening the velcro on the tape.

**Hand holder adjustment:**

- Insert the two threaded nuts(1) into the cut that matches the forearm so that it is in the plane of the hand grip.
- Place the hand holder (3) on the nuts and fasten both screws (2) to the first resistance.
- Adjust the length by sliding the hand holder (3) along the cut, then tighten the screw (2) to lock the position.

**Fitting instruction:**

- Locate the hinge pivot center over lateral epicondyle (outer elbow) and assess length.To adjust proper length press button (B) and slide the cuff away from the hinge to desire length. The lower extension (forearm part) should extend up to the radial styloid process (wrist). The upper extension (upper arm part) should extend up to the most comfortable position at the arm pit.
- If regulation of the range of motion is necessary it should be done before fitting.
- In order to set the selected angular range of movement, press the lock and slide it so that the number indicates the selected angle is in the point mark on the safeguard. Same steps should be taken to set the flexion and extension. To secure the selected angular movement range of motion, the plastic clamp (attached to the brace) should be inserted through the hole in the locks.
- Secure the straps, starting with two straps closest to the elbow. Cuff adjust to the limb circumference. If necessary, you should trim the straps to secure fit. Remove the hook piece, trim the strap and reapply the hook piece.
- To take off the brace loosen all straps and cuffs.

**Maintenance:**

The straps and fabric pads should be washed up by hand using mild detergents. Do not chlorinate. Do not machine wash, wring gently before drying. Dry in extended position and do not iron.

**ATTENTION:**

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

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