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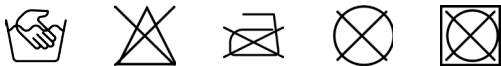
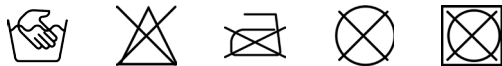
## Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

## Cleaning information

Elbow Safe should be washed using gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial warmth source. Dry in horizontal position. Do not iron.

## Guarantee

The guarantee lies within the framework of the legal regulations.

## Disposal

The product can be disposed of with the normal household waste.

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 mdh sp. z o.o.  
ul. Maratońska 104  
94-007 Łódź  
www.qmedbymeira.pl  
ortopedia@mdh.pl  
tel. +48 42 636 81 44

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EN

### Elbow Safe Elbow brace

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

### Product description

Highly elastic, breathable fabric ensures optimal adaptation to anatomical shape soft tissue compression which prevents oedema. Additional pads and band relieve muscle- tendon structures, especially attachments of muscles extensor and flexor. Elbow Safe is an universal aid for right and left elbow.

### Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. The product must not be worn on open wounds.