Wydanie: 21.09.2021
Edition: 21.09.2021
Ausgabe: 21.09.2021
Vydání: 21.09.2021

Producent/Manufacturer:

 MDH Sp. z o.o.
ul. Maratońska 104
94-007 Łódź
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.qmedbymejra.pl

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EN

Product description:

The Lumbo medical light active bandage is a highly effective medical product on the latest technical level. The anatomically knitted form offers a very good fit and high wearability. The compression of the bandage stimulates the metabolism and decreases muscle tension which brings relief and alleviation of pain. The four integrated stabilisation struts provide additional stability and support an anatomically correct posture. Safety instructions The first application of Lumbo medical light active bandage must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid congestion.

Employment according to purpose and indications:

The Lumbo medical Light is used in case of general backache, debility of the back muscles, lumbagos, irritations of the iliosacral joints, attrition of the spinal column. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

Application of the bandage:

1. Lay the bandage around the lumbar area and hold the left loop of the bandage in the stomach region.
2. Now pull the right loop over the left and fix the bandage by means of the Velcro fastening.
3. The bandage should have a steady, tight fit avoiding constrictions.
4. If necessary, readjust the fit of the bandage.
5. The metal struts can be adjusted in the back area if required. In this case please contact specialist dealer or the responsible physician.
6. The exchange of the centre metal struts by plastic struts must only be carried out by authorised personnel or the responsible physician.

Contraindications:

Provided that the product is employed properly contraindications are not known. If the bandage is too tight pressure symptoms may evolve.

Application duration:

The success of the bandage can be directly connected with the wearing period. According to latest investigations a permanent application does not weaken the muscles.

Further information:

- The active bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Lumbo medical light is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

Guarantee:

The guarantee lies within the framework of the legal regulations.

Storage information:

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information:

Remove the pad before washing. Please hang up the bandage dripping wet.

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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