Wydanie: 21.09.2021
Edition: 21.09.2021
Auszgabe: 21.09.2021
Vydání: 21.09.2021

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Producent/Manufacturer:

 MDH Sp. z o.o.
ul. Maratońska 104
94-007 Łódź
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.qmedbymejra.pl

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EN

Indications:

Dysfunctions of cervical area which do not require complete immobilization. Muscle and ligament injuries, strains and inflammation of muscular system, dysfunction of vertebral joints, injury of the intervertebral disc, osteoarthritis, rheumatic diseases, follow-up treatment of fractures.

Contraindications:

Stronger pain syndromes with deforming changes in the spine, discopathy of the cervical spine, postoperative conditions, fractures and dislocations of the cervical vertebrae.

Product description:

Light, one-piece, anatomically profiled construction made of polyurethane foam of carefully chosen density. Fastened and regulated with Velcro bands. The collar lining has anti-bacterial and anti-mycosis properties. The polyamide stiffening insert ensures better stabilization.

Selection and fitting:

Proper application of the collar is as important as proper sizing for immobilization and patient comfort. Use velcro tape to tighten moderately and button up collar.

Maintenance:

Collar can be washed by hand in a very mild detergents at temperatures up to 35°C. Do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat in spread form.

ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

Size:

		NECK CIRCUMFERENCE (CM)					
COLLAR HEIGHT (CM)		29,5 – 32,5	32,6 – 35,5	35,6 – 38,5	38,6 – 41,5	41,6 – 44,5	44,6 – 47,5
	7,0	A1	A2	A3	A4	A5	A6
	8,5	B1	B2	B3	B4	B5	B6
	10,0	C1	C2	C3	C4	C5	C6
	12,0	D1	D2	D3	D4	D5	D6

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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