



NÁVOD K POUŽITÍ / INSTRUCTIONS FOR USE

Katalogové číslo/Catalogue number: DRQF0F Basic UDI-DI: 59017804DRQF0FLR

(CZ)

Prohand **Stabilizační ortéza zápěstního kloubu**

Děkujeme, že jste si vybrali výrobek společnosti Meyra-Ortopedia Group. Výrobek splňuje všechny požadavky dle zákona o zdravotnických prostředcích.

Přečtěte si prosím pozorně návod k použití a dodržujte uvedené rady a doporučení. V případě dalších dotazů se prosím obraťte na odpovědného lékaře, specializovaného prodejce nebo naši servisní horkou linku.

Popis výrobku

Ortéza je vyrobená z prodyšné, elastické tkaniny. Zajišťuje optimální kompresi zápěstí pro zlepšení jeho stabilizace a zabránění otoku. Ortéza je k dispozici v univerzální verzi vhodné pro pravou i levou končetinu. Jednoduchost konstrukce umožňuje snadné použití při zachování lékařských vlastností a funkčnosti.

Bezpečnostní instrukce

Ortézu byste měli vždy používat podle pokynů svého lékaře nebo fyzioterapeuta. Ortézu by měl poprvé nasadit a upravit zdravotnický personál vyškolený na tento druh výrobků, jinak nelze zaručit odpovědnost za výrobek. Výrobek se nesmí nosit na otevřených ranách. Před použitím ortézy si pečlivě přečtěte návod.

(EN)

Prohand **orthosis stabilizing the wrist joint**

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

Product description

Elastic, breathable fabric provides optimal wrist compression, stabilization and prevents oedema. Prohand is available in one universal version, suitable for right and left hand. Due to simple construction it is easy to use and maintains proper medical and functional characteristics.

Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. The product must not be worn on open wounds.

Výrobce/Manufacturer

 mdh sp. z o.o.
ul. Maratońska 104
94-007 Łódź
www.qmedbymeira.pl
ortopedia@mdh.pl
tel. +48 42 636 81 44

