POZOR

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ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel.

Before the first use, please, read the user's manual carefully.

If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

Indications:

Wrist, hand and fingers disease which require support and stabilization in specific corrective and/or pain reducing positions. Tendon inflammation, carpal tunnel syndrome, rheumatic joint inflammation, contractures preventive treatment, circumferential nerves injuries, surgery.

Product description:

A splint for wrist, palm with a thumb hold. The splint assures the functional positions of wrist and hand. The soft hypoallergic pad provides for comfort, fasten with Velcro belts.

Use:

Please remember to choose the proper size of orthosis to have good treatment. Put hand and forearm on the splint and fasten the Velcro belts.

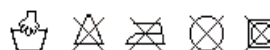
Maintenance:

The pad can be washed up by hand in lukewarm water using mild detergents. Do not chlorinate, do not whirlpool. Wring gently before drying. Dry in a distance of artificial sources of heat. Do not iron.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

	XS	S	M	L
Šířka dlaně* (cm) Width (cm)	6 - 7	7 - 8	8 - 9	9 - 10

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*hand width at the height of metacarpophalangeal joints

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