Qmed  
Care

HALLUSTOP

Qmed  
Care

HALLUSTOP

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**ENG**  
**Instructions for use**

Edition: AD240524  
Catalogue number: DRIE0A

Thanks to the use of straps made of elastic material, the orthosis restores the natural position of the toe reducing pain. The stabilising stripe restores the natural transverse arch of the foot by eliminating tension and discomfort in the mid-foot. The strong but highly flexible material guarantees a sense of comfort. Three-point support system designed for even better adjustment of corrective forces. Thanks to the use of straps made of elastic material, the orthosis restores the natural position of the toe, reducing pain. The stabilising strap and optional use of a metatarsal pellet restore the natural transverse arch of the foot, eliminating tension and discomfort in the midfoot. Durable and very flexible material guarantees a sense of comfort. The product is ideal for use during the day.

**Indications:**  
Prevention of the formation of hallux valgus.

**Contraindications:**  
Skin lesions around the area of exposure to the product.

**Usage:**  
Initially, use the product for about 1 hour a day, gradually increasing the time of use to 4-5 hours. The duration of use should be consulted with a doctor or physiotherapist.  
Maintenance: Wash with a damp cloth. Dry away from artificial heat sources.

**Size:** universal  
**Package contents:** 1 pc. (for right or left foot)

**Composition:**  
fabric: 80% polyester, 20% spandex,  
touch fastener: 100% nylon

**Intended user:**  
The medical device is intended for the general public (a non-professional user).

Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the Member State (applicable to the EU) where the user and/or patient is based.

**Disposal of the medical device:** The device must be disposed of following the applicable local environmental regulations.

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