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**USER MANUAL**  
**Elbow crutch**  
**Elbow crutch with two adjustment points**  
Edition 07.13

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**PRODUCT DESCRIPTION:**

The elbow crutches are made of high quality material with round cross section aluminium profile. They are equipped with the half-elliptical forearm suspender that guarantees perfect elbow stability and the handle (hand grip), designed to fit anatomical shape of a hand and covered with soft in touch, pleasant material that prevents corns creation and with non-skid attachment element. Both the suspender and the handle are made of PU plastics, of especially selected hardness. The adjustments of crutch length and handle height can be performed by steps 2.5 cm, by means of friction-latch type regulator, stabilized with additional fastener.

**USAGE:**

For persons who need assistance while walking, in order to suspend and move their body weight in case of injuries or orthopaedic disorders, such as fractures, strokes or paresis of lower limb, or while learning to walk. Furthermore crutches are suitable for elderly persons as a body balance support during walking.

**KONTRAINDICATIONS:**

Coordination disorders, balance disorders, injuries or traumas of upper limbs that results in unavailability to use crutches.

**MAIN FEATURES:**

1. Universal,
2. made of aluminium,
3. suspender and handle are made of PU plastics,
4. height adjustments in step of 2.5 cm,
5. maximum load in case of double adjustment point crutches: 100 kg / 140 kg (see the table below),
6. rubber made foot attachment: elastic, grey (does not stain floors or parkets),
7. cat's eye mounted in the handle.

**USING AND ADJUSTMENTS:**

Crutches are shipped in its minimum length (retracted). After removing from package, crutches need to be set to the desired length (height). It is recommended that the height of the crutch is set so that the handle levels a greater trochanter of the thigh bone.

**MAINTENANCE AND SERVICE:**

1. Aluminium profiles of the crutches are to be cleaned with a damp cloth.
2. Hand grip and elbow suspender are to be cleaned with soft detergents,
3. All moving elements of the crutches (friction-latch regulator) are to be removed immediately.

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**SAFETY OF USAGE:**

1. Crutches are intended for use indoors and outdoors, on various surface types.
2. When using crutches on a surface that does not provide proper support, such as wet, icy, greasy or sandy surfaces => one should be extremely cautious.
3. The crutches are designed to support weights of up to 100 kg / 140 kg (see the table below).
4. All the construction elements of the crutches should be periodically checked visually, especially:
  - friction-latch mechanism of the crutches,
  - non-skid rubber foot.
5. In case of any visible or sensible damages of the crutches (such as cracks), one should not use them, which may result in an injury.
6. It is recommended that all the repairs where done by the qualified and experienced service personnel of the distributor – see below for contact details.

For the product to maintain its proper characteristics, it is extremely important to maintain all its elements clean at all times and store it properly. Prolonged storing at premises of very high humidity (saunas, laundry rooms, etc.) or outdoors, along with lack of adequate maintenance, will lead to quick loss of the initial utility and aesthetical properties.

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**DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVIS**

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