VITEA®
CARE

CE

POPIS VÝROBKU:

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PRODUCT DESCRIPTION:

The bottom part of the aluminum cane is in four segments structure, coated with a protective layer of varnish and equipped with non-slip rubber strap. An aluminium tube with polyurethane handle builds the walking stick upper part. The handle is slipped into the frame which has holes located every 2.5 cm. The holes enable the walking stick height regulation. A frictional-pawl regulator guarantees a required height.

USE:

For people who need support during walking, after lower limbs injuries, for balancing and body weight transferring in traumatic and orthopaedic diseases. Also for elderly people as a body balance support during walking.

CONTRAINDICATIONS:

Balance and motor coordination disorder, upper limb injuries and dysfunctions which preclude correct usage.

PRODUCT CHARACTERISTICS:

1. universal (for both left and right hand);
2. made of aluminium;
3. handle made of PU;
4. height regulation every 2,5 cm;
5. max user weight: 100 kg;
6. gum strap: elastic, non-marking
7. foldable;
8. color: black, brown.

REGULATION AND USE:

Product is packed folded. Take out of wrapping. Unfold to required height.

SAFETY OF USING:

1. The walking stick is designed to use inside and outside on various surfaces;
2. One should be careful using the walking stick on an insecure surface: wet, icy, greasy, sandy- risk of falling down;
3. Max user weight: 100 kg;
4. Technical survey should be done regularly, special attention should be paid to: frictional-pawl regulator function, the function of anti-slip strap.
5. It is unacceptable to use the walker after discovering any visible or detectable damages as it may lead to accident.
6. All the repairs needs to be performed by a skilled personnel.

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EVERYDAY USE AND CONSERVATION:

1. Clean the cane with a moist cloth;
2. Always use gentle detergents which do not damage the protective coat of the frame;
3. Clean regularly all the mobile elements (pawl blockade mechanism).

Keeping all parts of the walker clean and the way of preserving it, is essential for keeping its original parameters. Keeping the walker in a moist place (steambaths, loudries) or outside together with a lack of hygiene can cause a rapid lost of original parameters.

SPECIFICATION:

Height: 80 - 90 cm

Regulation: every 2,5 cm

Weight: 0,35 kg

Materials: Aluminium, PU, rubber

Serial number: on the product

Product durability: 3 years

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mdh sp. z o.o.

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90-349 Łódź

DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVIS

MEYRA ČR s. r. o.

Hrusická 2538/5

141 00 Praha 4 – Spořilov

tel.: 272 761 102

e-mail: meyra@meyra.cz