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**INSTRUCTIONS FOR USE**  
WHEELER  
WHEELED WALKER  
VCBP0032

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PLEASE READ CAREFULLY BEFORE  
USING WHOLE OF INSTRUCTIONS

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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1. Handle
2. The aluminum support frame
3. The reinforcing frame
4. The telescopic height adjustment
5. Submission button of a walker
6. The reinforcing frame
7. Legs finished rubber non-slip security
8. Legs finished wheels

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Walking aid designed for disabled people for those with locomotor disorders.

Ideal for rehabilitation process and physiotherapy (verticalisation, lower limb relief).

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves.

- The product is packed In folder position.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.

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- In order to unfold the walker, should leaning its sides maximally outside till the moment of latching blockades. (Illustration 3.)
- For easy storage and transport, the walker can be folded.
- To do that, should push the submission button placed on the frame and fold side

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frames.

- Release submission button. Walker should be folded to position shown on the Illustration 2
- Walker can be moved, holding by side frames.

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### HEIGHT ADJUSTMENT

- Before using a walker, should adjusted to the user's height by telescopically adjustable feet length.
- To do this, press the height adjustment pin of the legs and depending on the needs reducing the height of the legs or extend the leg (Illustration 4.)

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#### MOVING WITH WALKER WITH WHEELS

- In the process of using a walker, the person must move walker forward and take a step forward, walk into the frame.

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#### CARE AND MAINTENANCE

- Clean the walker frame with a damp cloth
- Remove contaminants on a regular basis
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly.
- Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic.
- Do not expose the walker on to extreme weather conditions (rain, snow very strong sun), because certain parts can change colour.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the walker with pressure or steam aggregates!

#### TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Every two weeks	Screw, nut and rivets control regarding their correct fixing.	Make sure if all the screws, top and rivets are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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Repair

- In case of reparation, please contact the producer.

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Customer service

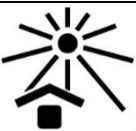
- If you have any questions or need help, contact the producer.

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