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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE!**

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Four-wheel walker ambona type is a modern, stable, durable and safe product. It was created to support the rehabilitation of the patient, allow the patient to move without help of the others.

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1. Armrest
2. Armrest height adjustment knob.
3. Caster rear wheel with locking rotation of the wheels.
4. Handle holder.
5. Handle holder mounting knob
6. Barrier
7. Upper stabilizing frame
8. Main frame
9. Lower stabilizing frame
10. Caster front wheel

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Elements that should be inside the packaging:

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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Ilustrace 10 / Illustration 10.

- The main framework for stabilizing the frame should be fixed by using four screws and washers (Figure 11, 12.)

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Ilustrace 12 / Illustration 12.

- In the lower part of the armrest are located eight holes for mounting screws.
- Bearer must be fixed in such a way that the knobs are positioned to the outer side in front of the support.

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Ilustrace 15 / Illustration 15.

- In bearer insert the handles and tighten the mounting knob handles.

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To unfold the rollator, should follow the opposite to the above manner of conduct

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BRAKE SYSTEM

- The walker has a braking system located on the rear wheels. The braking system is the locking device. In order to stop the walker and immobilize it press down the lock located on the rear wheels. To unlock the wheels, pull up the lock.

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OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

STAIRS

- When the patient climbs the stairs or down the stairs needs belaying of two people. One person should assist the patient, the other person should carry walker, because the patient after reaching the top will again need support. While moving the equipment, it is recommended to hold it for the frame.

CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.
- Do not expose the walker on to extreme weather conditions (rain, snow very strong sun), because certain parts can change colour.
- If walker has wetted in the rain, it must be dried, use anticorrosive to keep the components in good condition.
- Regularly check the mobility of the rotating elements, and if its need to apply grease.
- Avoid contact tires of the wheels with the acid to prevent the deterioration of the tire rubber quality.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

New user

The rollator can be passed to a new user after technical inspection performed by qualified medical personnel. The former user should provide complete technical documentation which is crucial for safety of using.

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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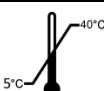
Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer/distributor.

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www.mdh.pl

www.viteacare.com

DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVICE

MEYRA ČR s.r.o.

Hrusická 2538/5

141 00 Praha 4

T: +420 272 761 102

www.meyra.cz