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USER'S MANUAL
Foldable cane
VCBP0022
Edition MG 08.2021

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**PLEASE READ CAREFULLY
BEFORE USING
WHOLE OF INSTRUCTIONS**

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The bottom part of the cane consists of steel, covered with chrome construction and equipped with four legs with non-slip gum strap. An aluminium tube with polyurethane handle builds the walking stick upper part. The handle is slipped into the frame which has holes located every 2.5 cm. The holes enable the walking stick regulation in a range from 80 to 90 cm, every 2.5 cm. A frictional- pawl regulator guarantees a required height.

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1. Universal (for both left and right hand);
2. Made of aluminium;
3. Handle made of PU;
4. Height regulation every 2,5 cm;
5. Max user weight: 100 kg;
6. Gum strap: elastic, non-marking;
7. Foldable;

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For people who need support during walking, after lower limbs injuries, for balancing and body weight transferring in traumatic and orthopaedic diseases. Also for elderly people as a body balance support during walking.

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Balance and motor coordination disorder, upper limb injuries and dysfunctions which preclude correct usage.

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Product is packed folded. Take out of wrapping. Unfold to required height.

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MAINTENANCE AND SERVICE:

- Clean the cane with a moist cloth;
- Always use gentle detergents which do not damage the protective coat of the frame;
- Clean regularly all the mobile elements (pawl blockade mechanism).

Keeping all parts of the walker clean and the way of preserving it, is essential for keeping its original parameters. Keeping the walker in a moist place (steambaths, loudries) or outside together with a lack of hygiene can cause a rapid lost of original parameters.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the cane with pressure or steam aggregates!

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TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the cane (so as to reveal any hidden damage).

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Repair

- In case of reparation, please contact the producer.

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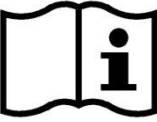
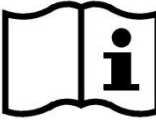
Customer service

- If you have any questions or need help, contact the producer.

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