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VCBK223 EDI

Verze MG 12.2015 / Rev. 02.2018

USER'S MANUAL

ROLLATOR WITH ARMREST

VCBK223 EDI

Edition MG 12.2015 / Rev. 02.2018

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VITEA[®]
CARE

CE

OBSAH / Table of contents

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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE**

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Four wheel rollator with footrest, equipped with two fixed rear wheels with brake and locking the wheels and the one front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Handles anatomical made of polyethylene, steal frame.

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Elements that should be inside the packaging:

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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HEIGHT ADJUSTMENT HANDLES.

- Handles should be located at a height that will provide comfort and a sense of security for the user. It is important to ensure that both hands are positioned at the same height.
- To change the height of the handles you should loosen the lever placed on the side of frame stand and choose the required height of the armrests. Screw the lever back thereafter. Always make sure that the both armrests are at the same height.
- After adjusting the appropriate height, tighten the knob (Illustration 8).
- Make sure that the handle is properly tightened, so that it prevents movement of the handles.

FOLDING:

- In order to fold rollator for ease transport, you must first block rotation of the wheels.
- Then it is necessary to remove the tray and basket for small items.
- Turn the blockade lever upward.
- Raise the transverse frame upwards.

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UNFOLD:

- Place your hand on the seat, push firmly and block the blockade (it is placed on the side of the frame) until its automatic latch. Please, make sure that the safety lever reached its ultimate position.
- Before each use, check the operation of the braking system. Squeeze the brake handles to check if the brake pads are right adhere wheels and stop the rollator.
- Press the brake handles down to activate the parking brakes and check if everything works properly.

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BRAKE FORCE ADJUSTMENT

- Brake force can be regulated – undo the thumb nut which is by the Bowden cable or tighten up a thumb nut (Illustration 12.).
- Brake shoes cannot touch the ring of a wheel.

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BRAKE SYSTEM

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake.
- To use the standard brake, pull the handle towards your body.
- To use a parking brake press the brake handle forward and you will hear the characteristic "click" of the handle latch. In order to release the brake pull the handle.
- Always use both the right and left brake in the same time.

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OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb you should set the rollator perpendicularly to the edge of the obstacle. Hold both handles, tilt the rollator in rear and raise the front wheels up.
- Then move the rear wheels to the edge of the obstacle, lift the rollator with handles slightly up and move rollator on obstacle.

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CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

New user

The rollator can be passed to a new user after technical inspection performed by qualified medical personnel. The former user should provide complete technical documentation which is crucial for safety of using.

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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Repair

- In case of reparation, please contact the distributor.

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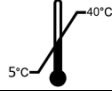
Customer service

- If you have any questions or need help, contact the distributor.

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MDH Sp.z o.o.

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90-349 Łódź

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DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVIS

MEYRA ČR s. r. o.

Hrusická 2538/5

141 00 Praha 4

tel.: +420 272 761 102

www.meyra.cz