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**PLEASE READ WHOLE INSTRUCTION  
CAREFULLY BEFORE USE.**

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## Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

## Product description

The shower stool is a light construction made of warm in contact polyurethane and aluminum. Stool legs of regulated height assure stabile support. The seat has a function of revolution which allows the patient to rotate in any direction without stand up.

### STOOL'S EQUIPMENT:

1. Polyurethane seat,
2. Aluminum legs,
3. Telescopic regulation of height,
4. Rubber straps on the top of stool's legs

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### **Application**

Hygiene activities with the use of bath or shower;  
Providing a patient with more safe position;  
Significant or partial limitation of movement  
(dysfunctions of motor system, problems with  
body balance and coordination)  
Designed for patients in serious conditions,  
elderly patients,  
Protection against wound infection or dipping,  
Ideal for hospitals, homecare.

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#### Contraindications

Contraindications to a sitting position, the lack of trunk stabilization. Patients with coordination disorders, balance and intellectual disabilities should not be left without supervision.

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#### Use

- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the stool until all parts are properly connected.

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## Safety of use

### WARNING !

- a) Before using the cushion, refer to this manual and keep it for future reference.
- b) The shower stool is designed to be used on flat, straight surfaces.
- c) The use of shower stool on the rough surface (of the slope more than 5%) may result in falling down and injury.
- d) It is strictly forbidden to stand on the stool.
- e) Maximum load of the stool – 110 kg.
- f) Use the stool in case of observing any visual or detectable damages (such as frame split) because it may result in falling down.

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## Everyday use and maintenance

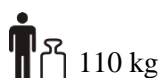
1. Clean the shower stool frame with a moist cloth.
2. Do not use sharp cleaning tools (risk of damaging the protective cover).
3. Clean with gentle detergent.
4. Regularly clean the shower stool's mobile elements (pawl blockade mechanism).

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Keeping all the elements clean is essential for preserving the product working parameters. Keeping the walking stick for a long time in a very moist room or outdoor may result in loss of esthetic and functional parameters of the product.

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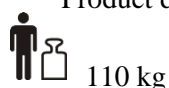
## Technical parameters

Height: 43-53 cm  
 Width: 43 cm  
 Seat diameter 28 cm  
 Weight: 2,2 kg  
 Materials: Aluminum, polyurethane, rubber.  
 Serial number/ production date: on the product  
 Product durability: 5 years from production

### Repair

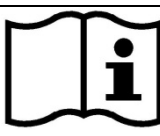
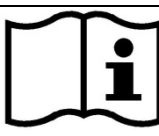
- In case of reparation, please contact the producer.

### Customer service

- If you have any questions or need help, contact the producer.

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