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VYDÁNÍ: 08.2021

VITEA[®]
CARE

USER'S MANUAL RAISED TOILET SEATS VCTP0031

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Product description:

The raised toilet seats with fixing to a toilet, lightweight, easy to clean, resistant to water, seats are made of contact-warm polyurethane material. Seats have screws clamp to a toilet and rubber pads that ensure stability. Three versions are available as pictured above.

Indications:

Support equipment, facilitating - enabling handicapped and elderly persons to use the toilet.

Contraindications

No possibility of adopting a sitting position. For safety reasons, people with balance disorders should not remain without supervision when using the equipment.

Usage:

The product in its original packaging folded, ready to be mounted on the toilet. In the version with handles, they should be mounted in the raised toilet seat beforehand. The product placed directly on the toilet seat and fasten by tightening the screw in the front of the product. Each time you use the product clean it with a damp cloth, using a mild detergent.

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Assembly:

1. Always be sure that the seat is correctly and securely locked in place before using. As shown in the illustrations below.

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2. Be sure that the adjustment knob in front of the seat is always tight.

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3. This product is designed for use with the most common types of toilets. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

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www.mdh.pl

www.viteacare.com

4. On models with arm rests: The arms are for the purpose of aiding when getting on and off the seat. The seat is contoured to provide safe and comfortable elevated seating. Removable handles are padded with a soft foam that provides a secure nonslip grip for added stability when lowering or rising from the seat.

Max load capacity – 100 kg

It is unacceptable to use a product that has not been firmly attached to a toilet – risk of falling.

DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit.

Technical parameters:

Height: 12,5 cm

Colour: white

Materials: polyurethane, rubber

Serial number/production date: on the package

Durability: 5 years from date of sale

Manufacturer:

mdh sp. z o.o.

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94-007 Łódź, Poland

tel. +48 42 674 83 84

fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com