Qmed®
Active

ZESTAW TAŚM DO ĆWICZEŃ - PĘTLA
RESISTANCE LOOP BANDS SET
JUEGO DE BANDAS ELÁSTICAS - BUCLE
ÜBUNGSBAND-SET - SCHLAUFE
КОМПЛЕКТ ЛЕНТ ДЛЯ УПРАЖНЕНИЙ – ПЕТЛЯ
GUMISZALAG HUROK SZETT (5DB, 50X5CM)
SADA POSILOVACÍCH GUM

mdh Sp. z o.o.
ul. Maratońska 104
94-007 Łódź, Polska (EU)
tel. +48 42 674 83 33
e-mail: port@meyragroup.com

www.qmedplus.pl

PL

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Wydanie: AD240524

Numer katalogowy: DRQM3M

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Edition: AD240524

Catalogue number: DRQM3M

- Suitable for all fitness levels
- Perfect for home and gym use
- Effective stretching and strengthening of all upper and lower muscle groups

A set of five coloured bands of varying resistance for all-round functional training and resistance exercises. Perfect for all fitness levels, easy to use at home as well as in the gym or Pilates classes. It provides effective stretching and strengthening of all muscle groups of upper and lower limbs. It also helps to increase joint mobility.

Resistance levels:

Yellow - 4.5-7kg

Red - 9-15.5 kg

Blue - 13.5-20 kg

Green - 18-27 kg

Black - 22.5-31.5 kg

Indications:

- rehabilitation exercises
- increasing joint mobility
- general development exercises
- rapid increase in muscle strength
- TRX training, Pilates

Contraindications:

- Do not use in patients with sensory disorders or skin lesions such as ulcers, blisters, abrasions, etc.
- For post-injury strengthening exercises, perform the exercises under the supervision of qualified medical personnel.

Note!

Perform a warm-up before training Prepare the training area appropriately Adjust the resistance of the band to your ability Do not forget to breathe during exercise If pain occurs, discontinue use of the bands.

Maintenance:

Wash with lukewarm water with soap. Dry away from artificial heat sources. Put some talcum powder before reuse.

Composition: 100% natural latex

Size: 50x5 cm Suitable user:

The medical device is intended for the general public (a non-professional user).

Disposal of the medical device: The device must be disposed of following the applicable local environmental regulations.

Caution: Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the Member State (applicable to the EU) where the user and/or patient is based.

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