Popis výrobku

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Vydání: [07.2021]

Model: DRQM6H

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Version: [July 2021]

Model: DRQM6H

- Rated power: 5W
- Voltage: DC5V=1A
- Charging time: 2H
- Use time: 60 minutes
- Time for one cycle: 5 minutes
- Dimensions: 260x52x90 mm
- Material: ABS+ silicone
- Sound intensity: 35 dB
- Charging method: micro USB cable
- Built-in Lithium battery
- Battery Capacity: 2200 mA
- Application: massage of the neck, back, abdominal, limb and feet muscles

The product complies with:

EN 55014-1: 2017

EN 55014-2: 2015

EN IEC 61000-3-2: 2019

EN IEC 61000-3-3: 2013 + A1: 2019

Convenient and user-friendly massage is at hand in the comfort of your own home. With the help of a stylish and elegant vibrating massager you can easily relax your shoulders, back and other parts of the body. The massager's shape and weight make it extremely handy and allow the user to perform the massage on different parts of the body, including even the back. Silicone cover of the massaging head with sufficient flexibility provides soft contact with the skin.

Employees spending long hours in the forced sedentary position at work, as well as everyone suffering from insomnia caused by nervous tension, or feeling muscle stiffness and fatigue after sports training or yoga session will appreciate Qmed vibrating massager. It perfectly fits your baggage while taking a trip.

With a carefully developed construction and advanced technology, our vibrating massager combines safe operation and modern design,

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which make it a perfect gift for those who love fashionable, high quality products.

Manual:

When the device is off, pressing the  button switches on the massager in the slowest operation mode and lights up the blue LED. To switch off the device and LED, short-press the  button again.

When the device is on, short-press the  button once to accelerate the device to the medium operation mode, or twice to advance to the fastest operation mode.

To activate auto 1 or auto 2 mode, long-press the .

If both buttons are simultaneously pressed for 3 seconds, the current operation mode will be recorded. To remove the recorded mode from the memory, press both buttons for 3 seconds again.

Blinking LED signals low battery level. Connect the massager to the energy source. During the charging the LED continues to blink.

CAUTION: When using the device always take the basic safety precautions specified in the information below. Read the manual carefully before using this product. Proper use of this device is recommended to achieve the best results. Improper use of the massager can result in injury or health complications. Keep this manual for future reference. The manufacturer reserves the right to modify the product specification without notice.

Do not use the product in case of:

- osteoporosis or diabetes
- heart diseases or when using electronic medical devices e.g. pacemaker
- neurological or perception disorders
- acute morbidities
- pregnancy or menstruation
- epidermis and dermis abrasion or damage

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- age below 12
- physical or psychological problems that prevent a person from using the product as intended
- general feeling of discomfort

Safe use instructions:

To reduce a risk of electric shock, fire or injuries, use the massager as instructed below:

1. Press the device gently against the skin and move it over the target area. Maximum massage time for a given part of the body is 60 s.
2. Do not apply the massager on the wet body. Never use the massager that has fallen into water.
3. High speed and strong pressure can be only applied to soft parts of the body. Avoid pressing the device strongly against the head and other osseous areas of the body.
4. Use the speed mode that is suitable in terms of the massage needs of a given part of the body.
5. Frequent using of high speed and applying strong pressure against a given part of the body can result in skin abrasions.
6. Never let the massager fall. The device can get damaged when hit or dropped.
7. Never use the device near the water.
8. Never use the massager contrary to the instructions.
9. Use the original battery charger exclusively.
10. Inspect your massager carefully before each use.
11. Do not introduce any modifications to the device
12. When charging the device do not run it or leave it unattended.
13. Do not leave the massager in the immediate vicinity of open flames.
14. Never cover the massager when it is switched on.

Warning:

1. Do not use the device near the water. Use and store the massager away from liquids and heat sources.

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2. After 1 hour of operation switch off the massager. Wait 30 minutes before continuing.
3. Disconnect the device from the battery charger when the charging is completed. Do not use the massager while charging.
4. When not consulted with the physician, it is forbidden to use the massager on the injured parts of the body.
5. Never let the children or persons unable to operate the device to use the massager.
6. Do not use the massager for the purposes other than the massage itself. This can result in an accident or injury.
7. Keep the massager away from children.

Maintenance:

1. Check if there is any damage or wear by visual inspection.
2. Be sure to remove the AC plug from the electrical outlet before cleaning the massager.
3. Wipe the surface of the device with a slightly damp towel and dry with a soft cloth. Make sure the massager is dry before putting them back in their packaging. Do not use solvents, toilet cleaners, sharp brushes and hard objects and caustic cleaners. Do not attempt to clean internal surfaces. Disassembling the device to clean internal mechanisms could result in device failure.
4. When storing or traveling for a long time, separate the device from the battery and place it in the device's original box.
5. It is extremely important to keep all the massager components clean to maintain the product parameters.
6. Storing the massager in areas with very high humidity (baths, laundry, etc.) or in the open air as well as lack of hygiene causes rapid loss of operating parameters and aesthetic of the product.

Disclaimer:

Before using the device read the manual carefully and use it as instructed. MDH sp. z o.o. shall not be held liable for any damage or loss suffered by the customer due to improper use of the device.

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The product complies with the requirements on electric and electronic equipment specified in the relevant standards. It is not to be disposed of with other household waste.

Following the end of the device's service life, the user is obliged to hand the device over to a relevant collection point and failure to comply with this obligation may result in legal sanctions being applied, as provided for in the waste regulations in force. For more information on the collection systems available, please contact your local waste disposal center.

Manufacturer:

mdh Sp. z o.o.
ul. Maratońska 104,
94-007 Łódź, Poland
tel. +48 42 674 83 33
e-mail: profilaktyka@mdh.pl