+REHAB®

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Due to the unique shape of the QMED Seat ring cushion, it immediately relieves, and with its anatomical design it contributes to better comfort, even in long-term sitting positions. Deepening under the thighs reduces pressure on the thighs and promotes better blood circulation in the legs. The purpose of the hinder recess is to relieve the lower part of the spine, the tail bone. The hole in the middle of the cushion relieves pain on the sensitive areas during seating, while preventing it from coming into direct contact with the surface of the chair, thus saving it from irritation. The seat surface of the cushion slopes forward at a slight angle, so it also carries the spine-saving properties of the wedge cushions and helps to adopt the correct posture while sitting.

Indication: External and internal hemorrhoids; Rectal cracks; Tail bone, pelvic pain; Inflammation of the prostate; Prevention of decubitus; After child-birth and altar surgery;

The anti-slip cover of the seat ring keeps the cushion in place during use, making it easy and reliable to use on any kind of seat. Can be used at home or in an office chair, car or wheelchair. It has a zippered, removable and washable, breathable and antiperspirant cover.

Composition: Cushion: 100% polyurethane;

Cover: Side fabric: 100% polyester; Bottom fabric: 55% polyester, 45% PVC

Cleaning: Wash the pillow cover with hands only in max. 30 °C water. Do not clean the pillow in a washing machine or dryer!

Attention!

The cushion is not a medical device. Do not use in case of loss of feeling.